

Headlight Clip Mounting Instructions



M-CZ

Mounting Clip for Carl Zeiss® Prismatic Loupes (3.25x and higher)

M-CZ


For use with: Zeiss® prismatic loupes
(3.25x and higher).

Note: This product will not work if Zeiss® loupes are Gallilean style (less than 3.25x).
Call your supplier for the correct mount.


Mounting Instructions:



STEP 1: Install (2) #0-80 custom shoulder screws to front of Zeiss® loupes. Note: the orientation of the screws is not important at this time.

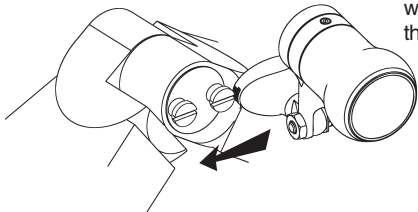


STEP 2: Line up the slot on the side of the headlight clip with the mounting screws as shown then slide the headlight on as shown.

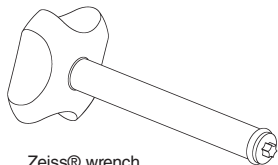

Zeiss® wrench
(provided with Zeiss® loupes)

STEP 3: Use the Zeiss® wrench (provided with Zeiss® loupes) to loosen the multi hinge on the loupes.

STEP 4: Rotate the headlight until it is centered above the loupes. Retighten the Zeiss® multi hinge.

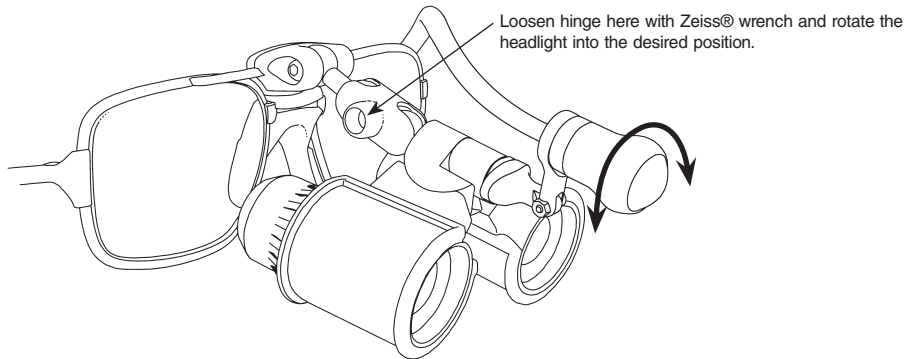


Loosen hinge here with Zeiss® wrench and rotate the headlight into the desired position.

STEP 5: The light guide should be routed around the loupe hinge and attached to the opposite temple piece with the supplied elastic clamp.

Cable Management


LED Cable: Use the provided molded clip and attach the cable as shown.

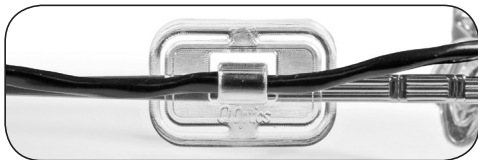

LED Cable: Use the provided spiral wrap and attach the cable as shown.



Fiberoptic Cable: Use the provided elastic strap and attach the cable to the arm of the loupes as shown.

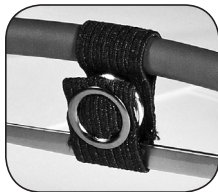

CE
0483

Q-Optics

A Division of **Quality Aspirators**

P.O. Box 382120 • Duncanville, Texas 75138 USA
1-800-858-2121 • (972) 298-2669 • Fax (972) 298-6592
www.q-optics.com

doc UI-317-10005-02
ecn 845