

Headlight Clip Mounting Instructions




M-QF ■ M-QFE ■ M-OR ■ M-SV ■ M-SU ■ M-KE
Mounting Clip for Flip Style Loupes

Q-Optics[®], Orascopic[®], Surgitel[®], SheerVision[®], Perioptix[®], Keeler[®]

M-QF, M-QFE, M-OR, M-SV, M-SU, M-KE

For use with: Q-Optics®, Orascopic®, Surgitel®, SheerVision®, Perioptix®,
Keeler® flip style loupes.

Mounting Instructions:

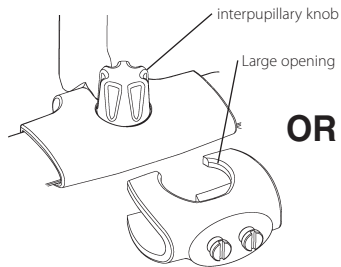


OR




STEP 1: Look inside mount to identify top of mount, or orient the large opening with the interpupillary knob if the mount is not labeled.


Note: Clip styles will vary from those shown in the instructions.

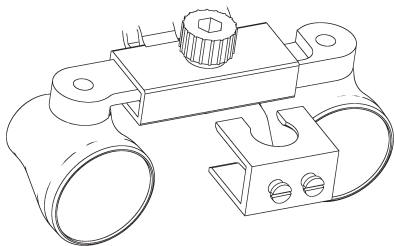

STEP 2: Slide the clip onto the front of the loupes with the slots in the clip surrounding the interpupillary knob of the loupes as shown.

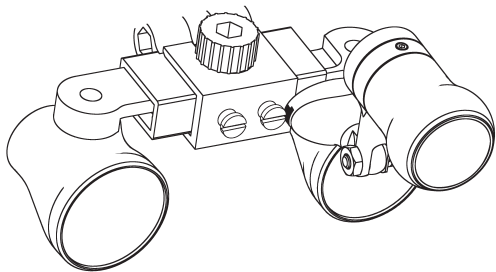


STEP 3: Line up the slot on the side of the headlight clip with the mounting screws as shown.

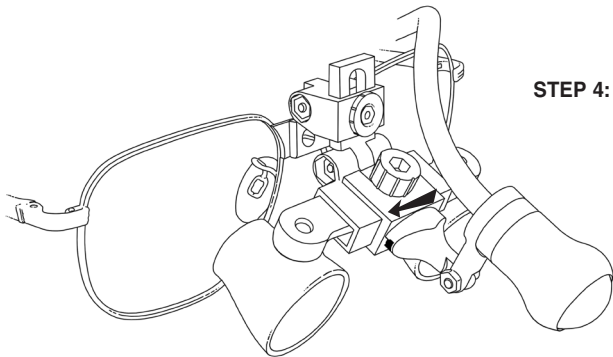


STEP 4: Slide the headlight on until it is centered on the mount as shown.

Cable Management


LED Cable: Use the provided molded clip and attach the cable as shown.

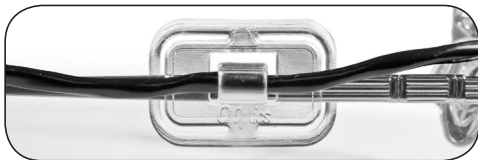

LED Cable: Use the provided spiral wrap and attach the cable as shown.

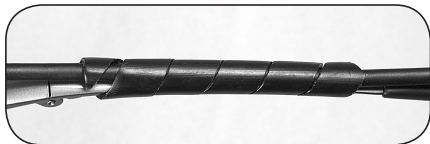

Fiberoptic Cable: Use the provided elastic strap and attach the cable to the arm of the loupes as shown.

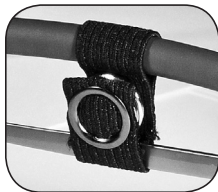

CE
0483

Q-Optics

A Division of **Quality Aspirators**

P.O. Box 382120 • Duncanville, Texas 75138 USA
1-800-858-2121 • (972) 298-2669 • Fax (972) 298-6592
www.q-optics.com

doc UI-317-10006-03
ecr 925