

Headlight Clip Mounting Instructions



M-SUTTLE
Mounting Clip for Surgitel® TTL Style Loupes


M-SUTTL

For use with: Surgitel® TTL style loupes.


Mounting Instructions:




STEP 1: Remove existing screw if necessary and slide mount onto clip as shown.

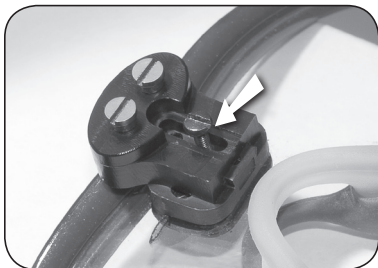

STEP 2: Insert screw thru slot and into the center hole of clip as shown. Adjust mount to desired position and tighten screw.

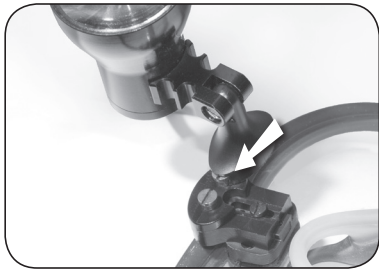


STEP 3: Line up the slot on the side of the headlight clip with the mounting screws as shown.

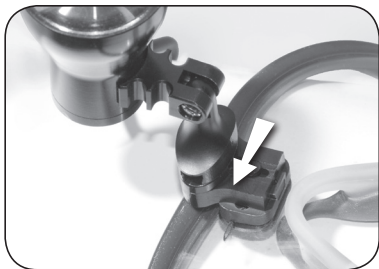


STEP 4: Slide the headlight on until it is centered on the mount as shown.

Cable Management

LED Cable: Use the provided molded clip and attach the cable as shown.

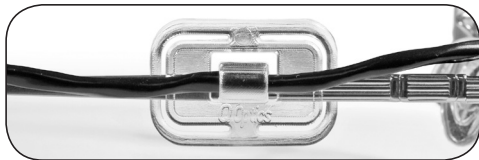

LED Cable: Use the provided spiral wrap and attach the cable as shown.

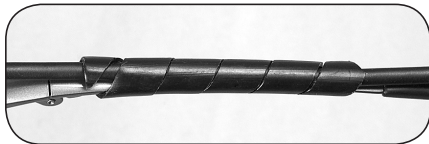

Fiberoptic Cable: Use the provided elastic strap and attach the cable to the arm of the loupes as shown.



CE
0483

Q-Optics

A Division of **Quality Aspirators**

P.O. Box 382120 • Duncanville, Texas 75138 USA
1-800-858-2121 • (972) 298-2669 • Fax (972) 298-6592
www.q-optics.com

doc UI-317-10014-01
ecn 925