


for Q-Optics®, Orasoptic®, Surgitel®, SheerVision® flip up loupes.
 Carl Zeiss® Galilean (less than 3.25x)

Q-COHL ■ SU-COHL ■ SV-COHL

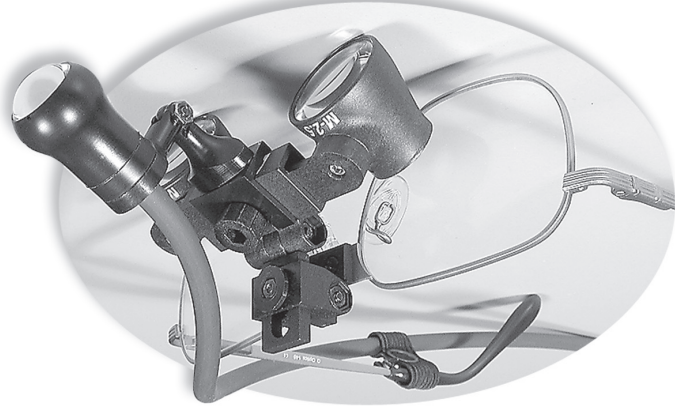


Quality Aspirators Headlight Clip Mounting Instructions

READ THIS FIRST!


CAUTION

Your light guide contains small **GLASS FIBERS** which **WILL BREAK** if the light guide is crimped, stretched, rolled over with a chair or stepped on. This breakage is not covered by your warranty. With use, some individual fibers will break and appear as small dots of light along your light guide. This is normal and should be of no concern. Severe breakage is indicated by large bands of light where the light guide has been crimped, crushed or otherwise mishandled. Your light guide will provide you with quality service as long as you supply the care and maintenance.

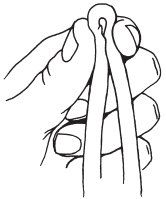

DO NOT
 CRIMP, STEP ON
 OR ROLL OVER
 THE LIGHT GUIDE
 WITH A CHAIR.

DO NOT
 STRETCH, PINCH
 OR CRUSH THE
 LIGHT GUIDE.

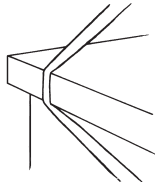

DO NOT AUTOCLAVE. Lens housing is **NOT** water-proof. Avoid immersing in liquid. Wipe light guide and lens housing with a damp cloth or antibacterial solution.



Quality Aspirators

P.O. Box 382120 • Duncanville, Texas 75138 USA
 1-800-858-2121 • (972) 298-2669 • Fax (972) 298-6592


Q-COHL, SU-COHL & SV-COHL

For use with: Q-Optics®, Orasoptic® flip up loupes, Surgitel® flip up loupes, SheerVision flip up loupes, Carl Zeiss® Galilean (less than 3.25x).


Mounting Instructions:




STEP 1: Look inside mount to identify top of mount.

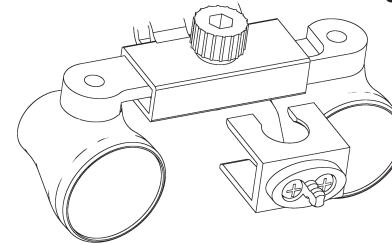

STEP 2: Slide the clip onto the front of the loupes with the slots in the clip surrounding the interpupillary knob of the loupes as shown.

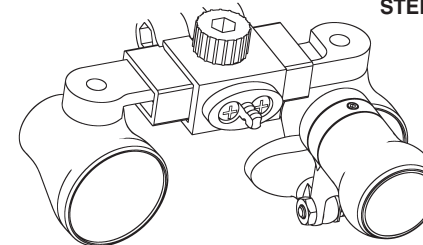

STEP 3: Slide the headlight and quick disconnect onto the clip as shown.

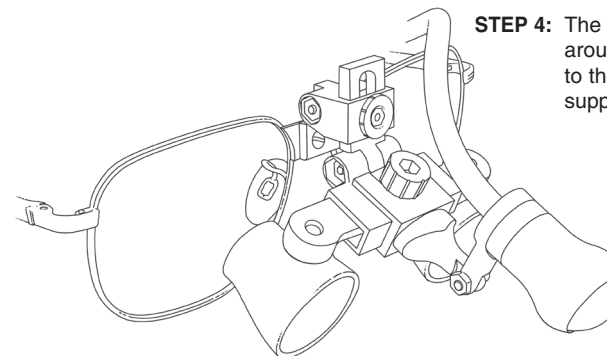

STEP 4: The light guide should be routed around the loupe hinge and attached to the opposite temple piece with the supplied elastic clamp.